**Instrucciones para instalar
la Mr. Hear Fitting Tool**

Sistema Operativo: macOS

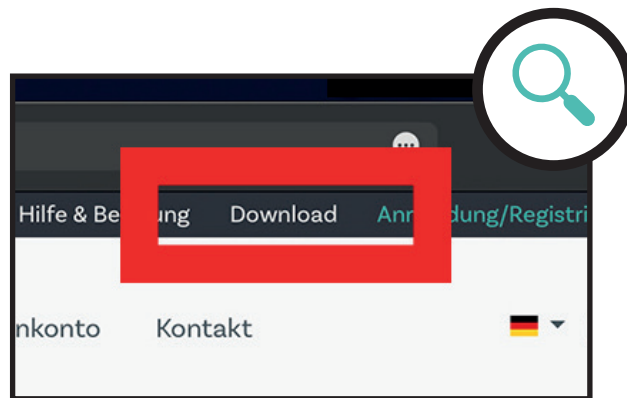
**Instructions for installing
the Mr. Hear Fitting Tool**

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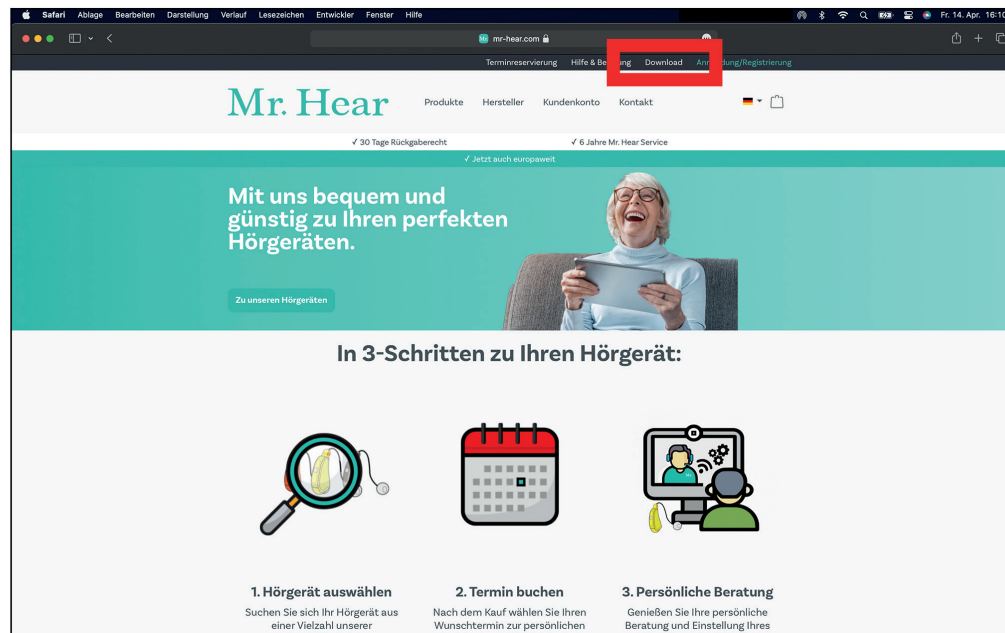
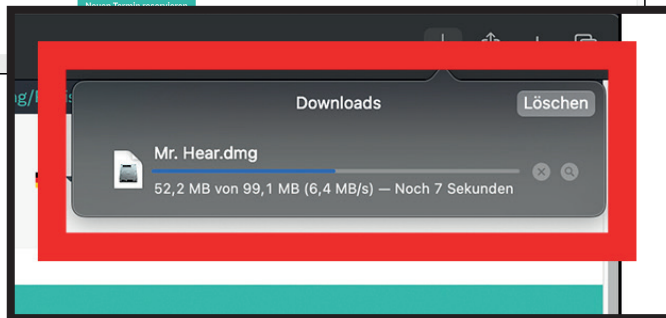
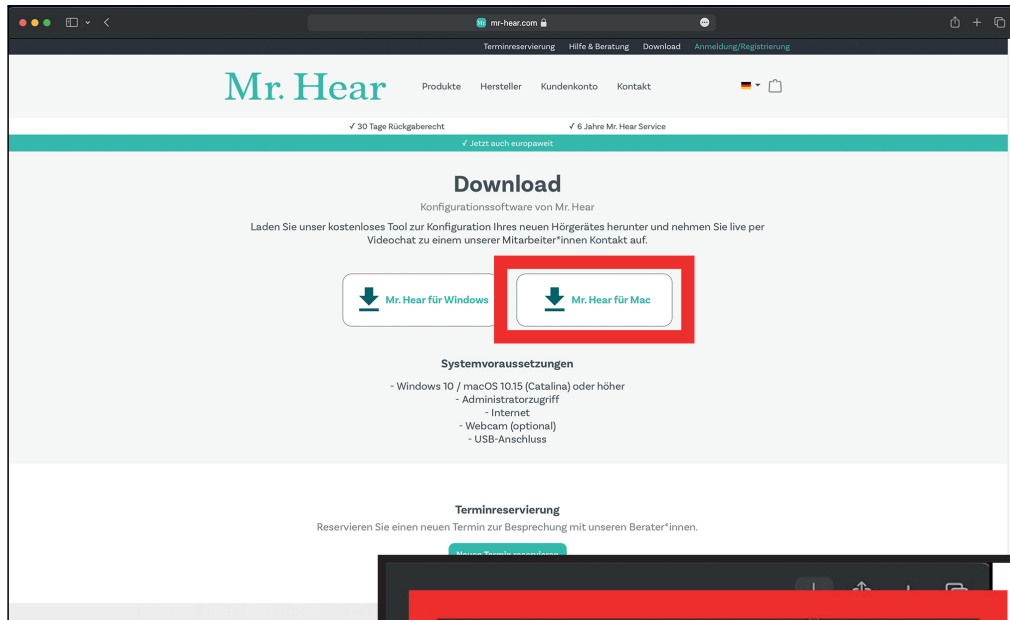
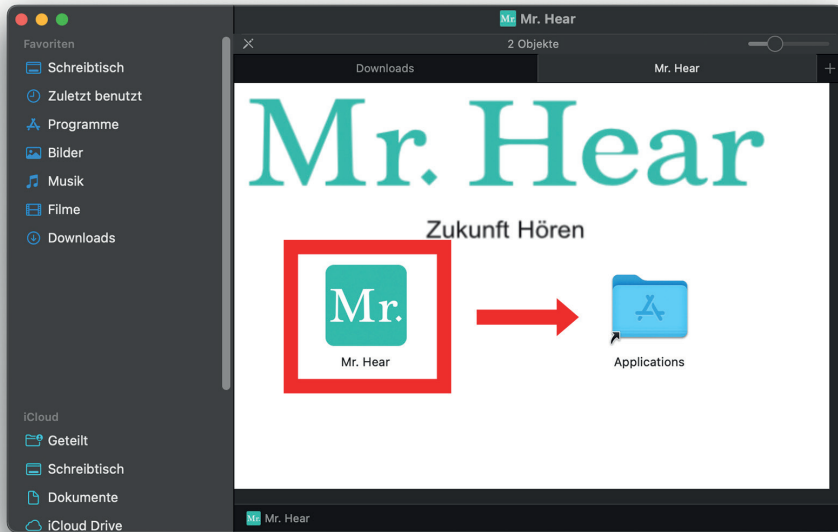
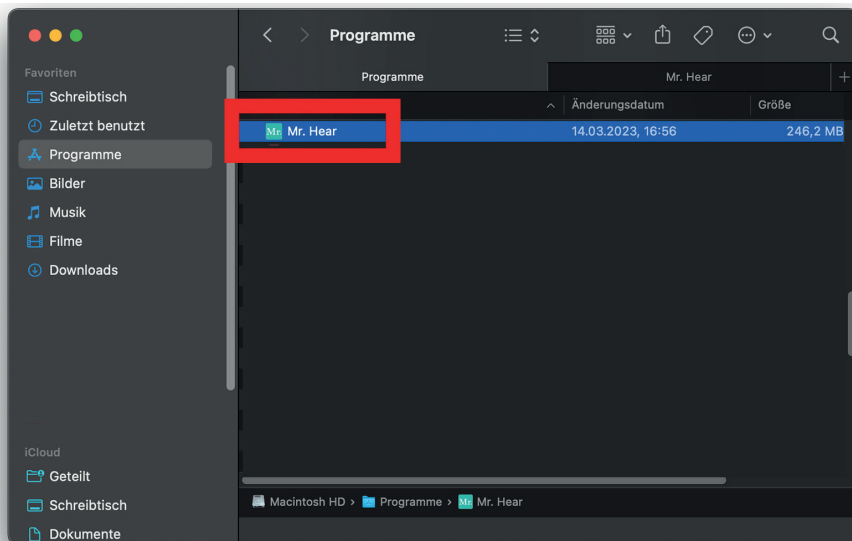
Step 3

Please open the downloaded file.
Drag the program while holding down the mouse button, as shown below, into the Programs/Applications folder.

Then, please go to the Programs folder and open the Mr. Hear Fitting Tool.

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Step 4

The Mr. Hear Fitting Tool will open. Log in with your customer account that you used to purchase the hearing aid/hearing aids and signed up with Mr. Hear.

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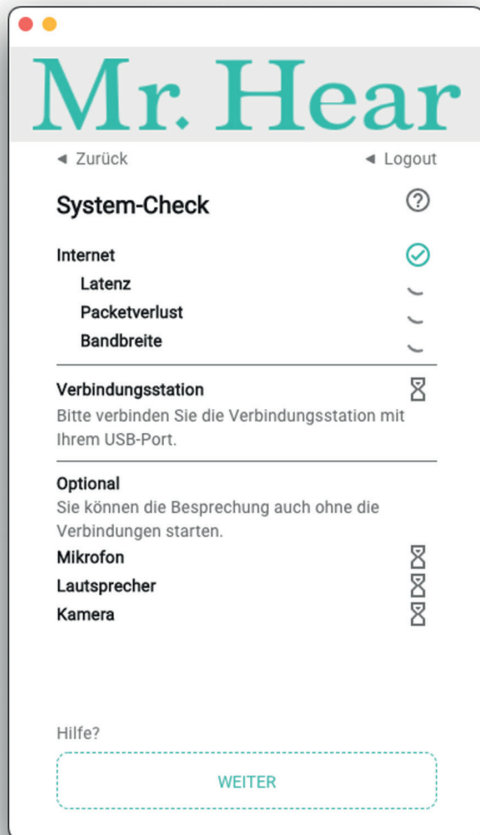
Step 5

Now you will see your booked appointment, which you have scheduled on our website, in Mr. Hear's program. Connect the provided Mr. Hear technology.

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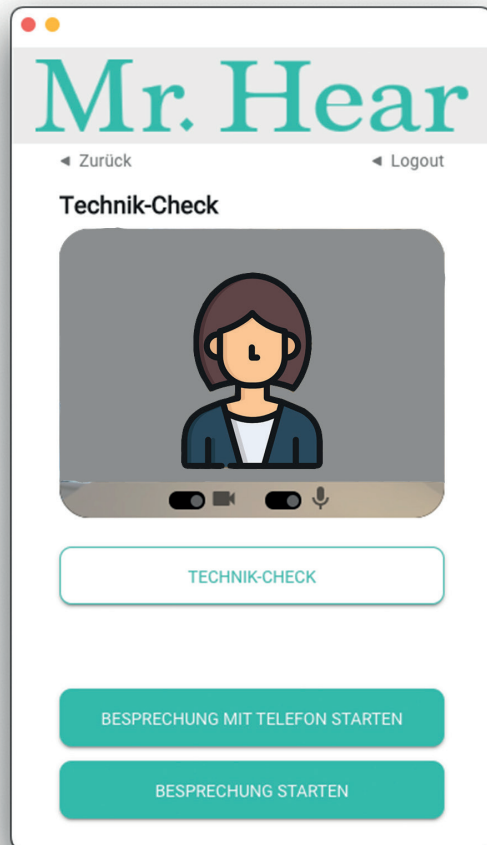
Step 6

Now an automatic system check will be performed.

Important: Please connect the provided Mr. Hear technology to your Mac before starting the system check.

Once the system check is complete, click on „CONTINUE“.

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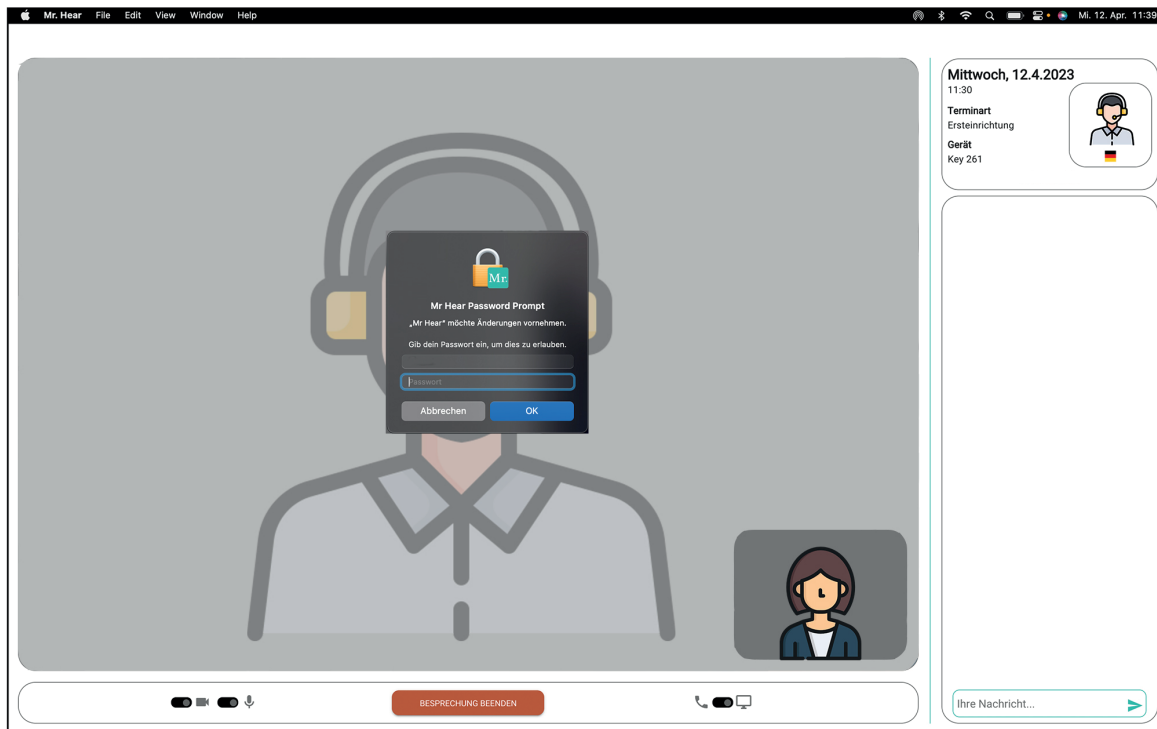
Step 7

In the tech check, you can test your webcam and microphone for functionality. If technical issues arise, click on „START MEETING WITH PHONE“. Otherwise, click on „START MEETING“.

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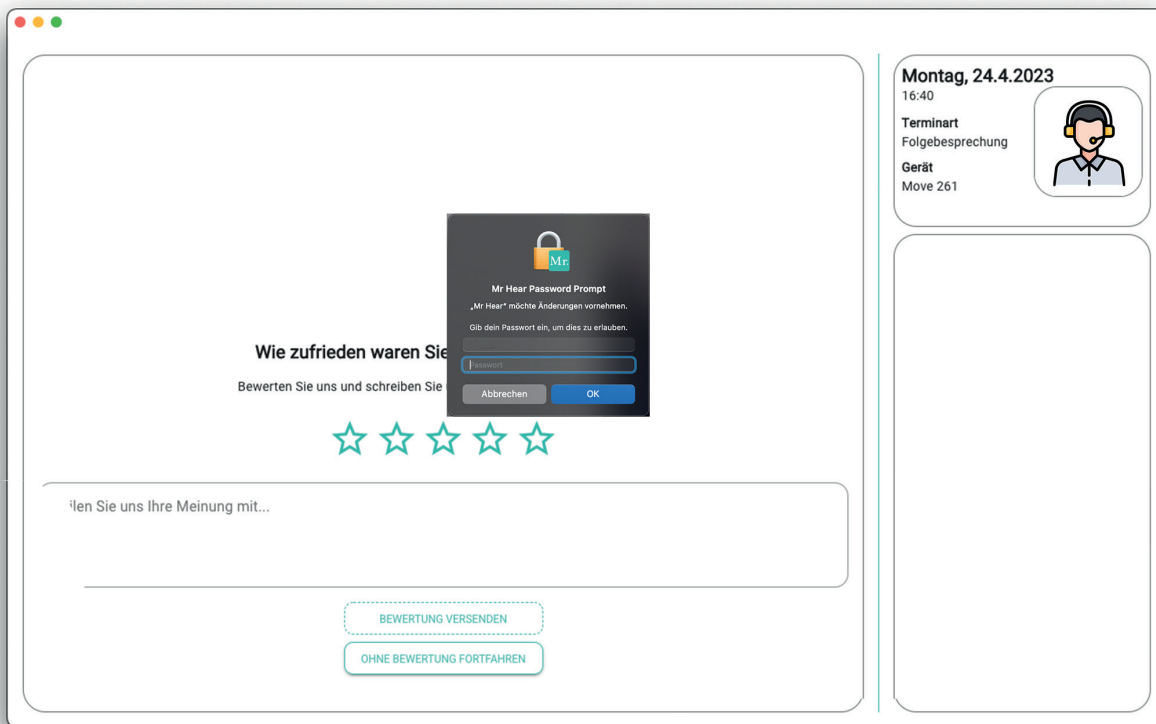
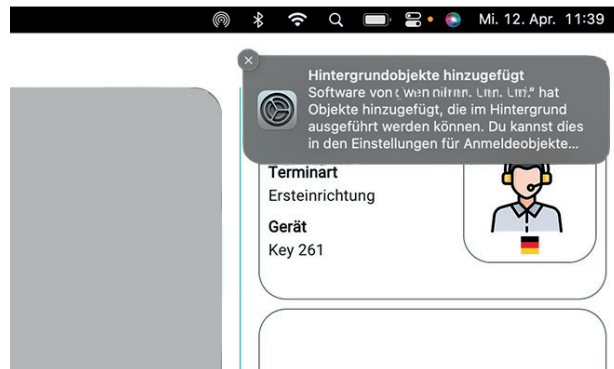
Step 8

After starting the appointment, your Apple Mac's password will be requested again

Please enter it and confirm with „OK“.
ATTENTION: Do not enter your Mr. Hear login here!

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Step 9

If everything is correct, a window will appear in the top right corner.

After your online appointment, your Apple Mac password will be requested again. Afterward, you can rate the conversation.

ATTENTION: Do not enter your Mr. Hear login here!